**tiq**<sup>®</sup>

# TENS

## HV6609

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## TENS WITH 4 CHANNELS

Thank you for purchasing our professional TENS HV-6609 ! It's a portable device used for therapy of acute and chronic condition - Rheumatism, Arthritis, Neck and Low Back Pain, Ankle and Foot Pain, Phantom Limb Pain and Muscle Strain.

### PLEASE READ THE INSTRUCTIONS BEFORE USING THIS PRODUCT

The device contains 4 output channels (2 pads per channel) that allow 4 body sites to be stimulated simultaneously. The output waveforms are in single-phasic square pulse, isolated between each channel. User can control the intensity for each channel separately.

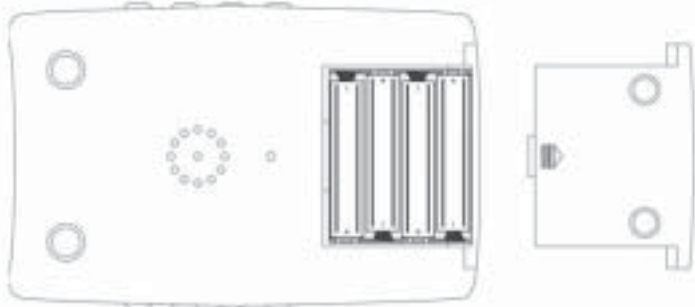
This unit can help to tone up the muscular masses and can be used for slenderizing, firming and relaxing muscles if use regularly.

## ACCESSORIES

- User Manual
- 4 lead wires
- 4 pairs of pads
- 2 pcs of pad holders
- 4 pairs of silicon gel pads

## BATTERIES INSTALLATION

1. Turn off power first if it is not the first time to use.
2. Scroll down the battery compartment cover and install four AAA batteries with correct polarity. When hearing "dick" "dick" sound, it indicates the unit is ready to use. Otherwise, re-install the batteries. **DO NOT** touch any key while installing the batteries.
3. Close the battery compartment cover tightly.

## WHAT'S TENS?

TENS is the short form for Transcutaneous Electrical Nerve Stimulation. It refers to the application of low-intensity and short duration pulses for pain relief. It's a safe, non-invasive, drug free method of controlling pain. Many clinical studies have proven the effectiveness of TENS for effective pain relief.

TENS therapy uses small electrical pulses transmit through the skin to the underlying peripheral nerves to help controlling the pain. It uses low frequency to stimulate the body to release its own pain relieving substances, called endorphins.

### PRE-CAUTION

Please do not use if you have any doubts on operating the unit. Consult your doctor or distributor for further instruction, if needed.

### DO NOT APPLY TO:

1. Pain of central origin, i.e. appendicitis, hepatitis.
2. Patients using Cardiac Pacemakers.
3. Patients with known myocardial arrhythmia without physician evaluation.
4. The carotid sinus nerves.
5. Laryngeal or pharyngeal muscles.
6. Undiagnosed syndromes until etiology is estimated.
7. Dermal diseases in the area being treated.
8. Internally (mouth).

### DO NOT USE IF YOU HAVE :

1. Heart disease.
  2. Pregnancy.
  3. Fever.
  4. High Blood pressure.
  5. Allergic skin.
- Keep the unit away from children.
  - Electrotherapy applied directly over a bone will cause pain.
  - Postoperative muscle contractions in the wound region may slow down the healing process.
  - Please stop to use the unit if redness and irritation of skin persist.

## FEATURES AND BENEFITS

- Four-Channel, Independent buttons controlling 50 levels of output intensity.
- Two power keys control two-channel groups: Left [  ] controls Channel 1 & 2 while the Right [  ] controls Channel 3 & 4.
- 12 pre-set programs (pulse frequency, pulse width, method and massage time is preset), 3 modes - Burst, Modulation and Conventional.
- 2 are user programmable (Conventional mode only)
- Large LCD displays Program, Intensity, Pulse Width, Count-down Treatment Time and Program mode.
- Treatment timer can be set from 1 minute to 23 hours and 59 minutes. Remaining treatment time is displayed on the LCD.
- The unit will be automatically switched off if not in use for 4 minutes.
- Load detection : The unit will stop ALL electrical pulse if no load is detected.
- Fast adjust protection: The intensity will stop at level 15 to avoid user increases the intensity too rapidly.
- Auto shut-off: LCD display will be automatically off if no operation is detected in 3 minutes.
- Low battery icon [  ] will be flashing if battery voltage is below 3.5V.
- User friendly - simply switch on, select mode, then adjust each output to obtain the desired pain relief.

## GETTING STARTED

### Preparation before treatment :

1. Connect the electrode wire to each output jack on the unit securely.
2. Attach a pad to each terminal lead.
3. Using a wet towel to clean up thoroughly the area for treatment.

### To Start Treatment:

1. Place the self-adhesive pads directly onto the area for treatment, above and below the area causing pain but never on top of the bones. The pads are self-adhesive, so there is no need for using tapes or gel.
2. Press hold [  ] or [  ] for 3 seconds to turn on the unit. Left button [  ] controls channel 1 & 2 and Right button [  ] for channel 3 & 4. The LCD then displays the initial default setting of treatment mode with intensity level at 0 and the timer set to 30 minutes.
3. Press [ PRG ] to select the program (refer to Table on page 14-16).  
For 2 user programmable programs, please read the procedure The 2 user programmable treatment setup at next section.

4. Press [ + ] key to start the program [ CH1 +/- och CH2 +/- ] at left side, [ CH3 +/- and CH4 +/- ] to the right. To adjust the intensity, press the [ + ] key to increase and [ - ] key to decrease.
5. When treatment starts, the  appear. If no load is detected, the unit will stop and  shows on the display. You can stop the treatment by pressing [ ON/OFF ], and  appear. You can restart by pressing [ ON/OFF ] again, and need to select the intensity again. You can also pause the treatment by pressing [  ] or [  ], and resume by pressing either [ CH1+ / CH2+ / CH3+ / CH4+ ].
  - For safety reason, the intensity go back to level 1 again, and you need to adjust it again.
6. Remove the pads and wires from the unit after used.  
Place them on the holder for next usage.  
Note : It's better to remove the batteries from the unit after use.

## THE 2 USER PROGRAMMABLE TREATMENT SETUP

1. Press [ PRG ] repeatedly until the LCD display P-C1 or P-C2.
2. To set your program, press hold [ PRG ] for 3 seconds. A SET icon will be displayed after "pip-pip" sound, the pulse frequency (Hz) digit will flash.
3. When it flashes, enter your favourite pulse frequency (Hz) by pressing relative [ + / - ] to adjust, then press [ PRG ] to confirm. To adjust the pulse width  $\mu$  and the massage time (Hr:Min), repeat the process again.
4. When finished the setting, press [  ] or [  ] once to exit the setup.

Note : User programmable treatment will be returned to default setting when turns-off.

## ELECTRODES PLACEMENT - SUGGESTED TREATMENT SITE

– see page 18-19.

## TREATMENT NOTIFICATION

The pads should be put on the area where it is painful or area around with the same direction of nerves normally. They can also be put on the area near the spine side where nerve mastering the area of where pain occurs.

The direction of pads can be parallel  or cross 

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**GB****STIMULATION MODE FOR EACH PRE-SET PROGRAM**

Program nr	Puls-1		Puls-2		Suggested massage duration, minute	Treatment mode	Suggested treatments
	Frequency (Hz)	Puls width (µS)	Frequency (Hz)	Puls width (µS)			
P01	70	200	110	100	30	Modulation	Relieve multi-pains
P02	150	30 (9 Pulses)	Burst Rate : 2Hz		30	Burst	Relieve post-surgical pain
P03	110	175	Not Applicable		30	Conventional	Relieve post-surgical pain
P04	2	50	10		30	30	Modulation Relieve chronic pain
P05	100	175	Not Applicable		30	Conventional	Relieve headache, calm nerve center
P06	10	250	100	250	30	Modulation	Relieve multi-pains
P07	2	30	Not Applicable		30	Conventional	Remove pain
P08	80	200	Not Applicable		30	Conventional	Relieve post-surgical pain
P09	35	200	Not Applicable		30	Conventional	Relieve chronic pain
P10	150	200 (9 pulses)	Burst Rate : 2Hz		30	Burst	Relieve pain
P11	35	30 (9 pulses)	Burst Rate : 2Hz		30	Burst	Relieve chronic pain
P12	80	30 (9 pulses)	Burst Rate : 2Hz		30	Burst	Relieve chronic pain
PC1	2 - 200	30 - 300	Not Applicable	30	Conventional		

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The pictures are illustration dirrerent  
relieve of pain.

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## GB TECHNICAL SPECIFICATION

Pulse amplitude	0-25V (200Hz@300¼S), or 0-120V (2Hz@30¼S) peak into 500 ohm load of each channel
Pulse rate (frequency)	2 Hz – 200 Hz, Adjustable
Pulse width (Duration)	30¼s – 300¼s, Adjustable
Wave Form	Single-Phasic square pulse
Modes	Burst, Modulation and Conventional
Battery	4 x AAA (Not included)
Battery life	Minimum 30 days (30 minutes play daily with output intensity =15)

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